

Bilicheck Meter Proficiency Checklist

Name:
Position:
Unit:
Date:
Unit Educator:

ACTIONS	DATE COMPLETED
1. Viewed BiliChek Video	
2. Read BiliChek Policy	
3. Performed three tests on self or staff member	
<input type="checkbox"/> Turns unit on.	
<input type="checkbox"/> Snaps tip into unit properly.	
<input type="checkbox"/> Performs calibration.	
<input type="checkbox"/> Removes the disposable cover appropriately in order replace for further testing.	
<input type="checkbox"/> Presses bilicheck tip flat onto forehead until light turns green and holds until light beeps and displays 004.	
<input type="checkbox"/> Removes from newborn forehead after each measurement.	
4. Errors	
<input type="checkbox"/> Knows how to refer to error code Quick Reference.	
<input type="checkbox"/> Knows how to clear error.	
<input type="checkbox"/> Documents result on newborn profile sheet.	
<input type="checkbox"/> Refers to policy to ensure results within normal limits.	
<input type="checkbox"/> If high result notify RN of attending physician.	
<input type="checkbox"/> If result >20 send serum level to lab.	
<input type="checkbox"/> Replaces disposable tip cover and places tip in package for future use.	
<input type="checkbox"/> Replaces meter in charger after use.	
5. Phototherapy	
<input type="checkbox"/> Places gel probe cover on infant prior to phototherapy initiation.	
<input type="checkbox"/> Turns phototherapy lights off prior to utilizing bilimeter.	