


Umbilical Artery Line

BP3 Kit Volume Measurement

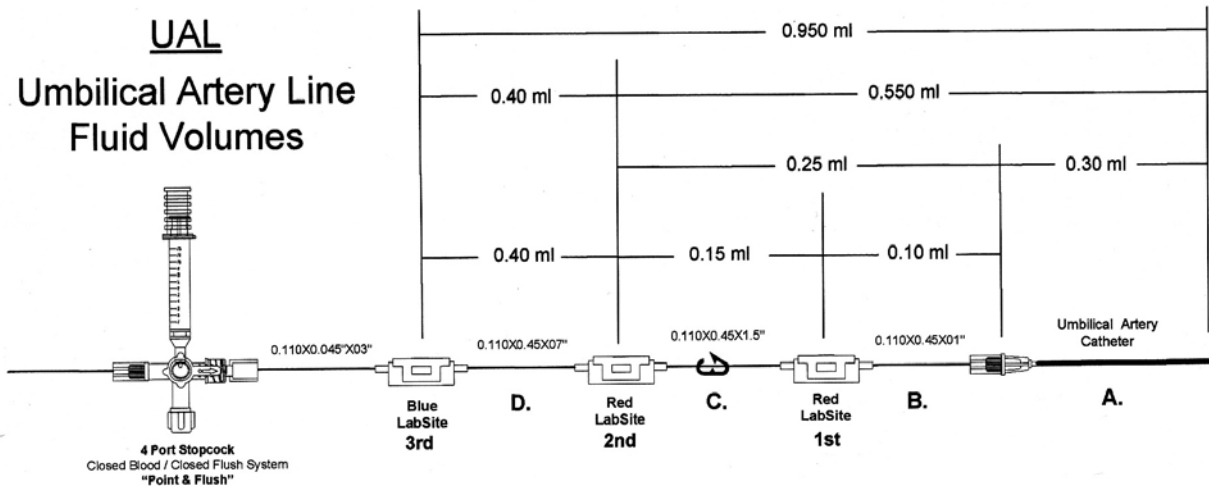

- A. Umbilical Artery Catheter (UAC) fluid volume = 0.08 ml to 0.30 ml depending UAC French size and catheter length.
 - B. Fluid volume from UAC connection to 1st LabSite = 0.10 ml
 - C. Fluid volume from 1st LabSite to 2nd LabSite = 0.15 ml
 - D. Fluid volume from 2nd LabSite to 3rd LabSite = 0.40 ml
- ⇒ 1st LabSite (red) – use slip-tip syringe to collect blood sample.
- Recommend using 1cc slip-tip syringe.
- ⇒ 2nd LabSite (red) – recommend using 3cc slip-tip syringe when collecting the waste / clearance during closed blood draw.
- Recommend withdrawing 1.5ml – 2ml waste / clearance.
 - Can be used with ML-100 to infuse medication / blood / fluids.
- ⇒ 3rd LabSite (blue) – can be used with ML-100 to infuse medication / blood / fluids.